

Steady Write

Writing Instrument



The Steady Write Writing Instrument by SP Ableware - Maddak is designed to help improve the handwriting of people with arthritis, Parkinson's or other hand limitations. The wide triangular base increases stability and balance, and guides the hand to smooth out shaky penmanship.

How to use:

1. Place the Steady Write base on the writing surface.
2. Grasp the Steady Write base (not the pen) and press down, moving the whole unit as you write.

Note: You may have to experiment with different ways of grasping the base to find the best method for you.

Dimensions: 7cm (H) x 7.6cm (L) x 5.1cm (W) (excluding pen) with .97cm pen holder diameter.



0800 212 333 www.c1south.co.nz