
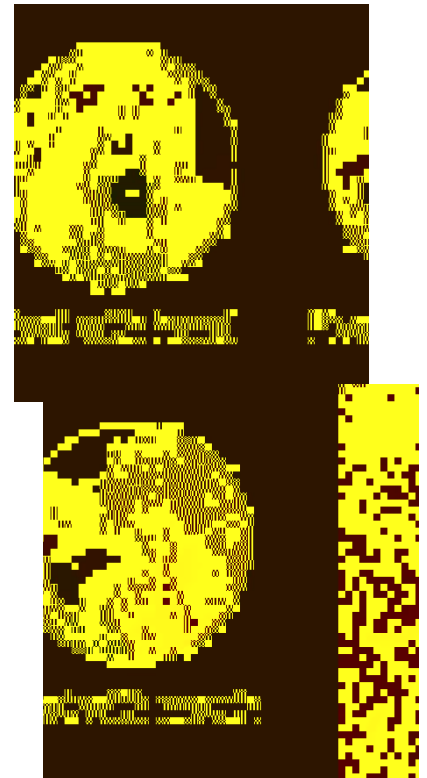


handSteady®

Now you can stop spills and socialise with confidence again

The **handSteady** cup with **Rotatable Handle and Hidden Lid** for people with tremor, pain, limited dexterity, weakness and other challenges.

- Cup stays upright and steady
- Tilts to drink by pivoting with mouth, thumb or other hand
- Lightweight, microwave and dishwasher safe
- Has discreet lid to fit a straw or to drink from

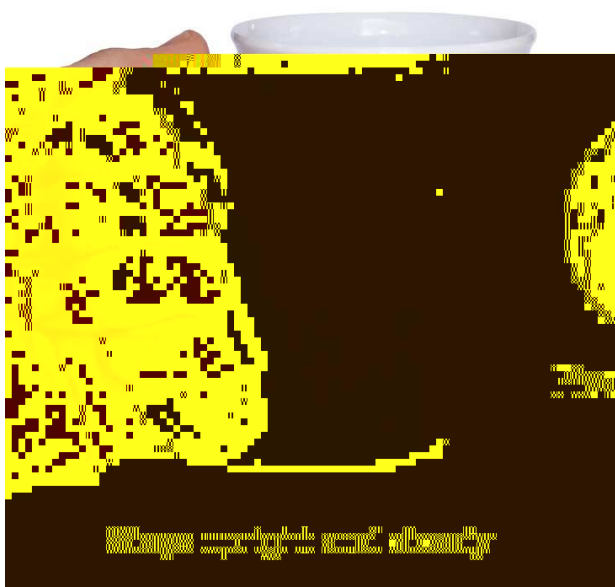

The handle smoothly rotates 360°



The lid is easy to insert and remove



Handle large enough for four fingers




The cup feels light when lifting




It's microwave and dishwasher safe

Step 1. Stays Upright


The Rotatable Handle™ frees the cup to stay upright and self-level using gravity. This reduces the chances of spills, frees you to hold the handle in comfortable and stable positions and frees you to use one hand rather than two if you have weak wrists.

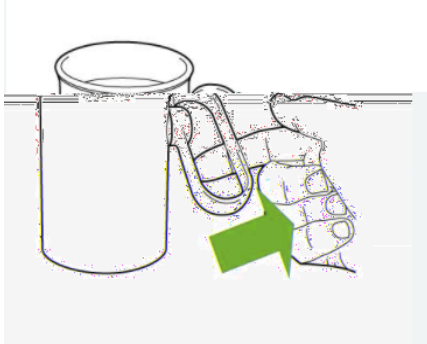

Tilt the handle to make it easy to pick up

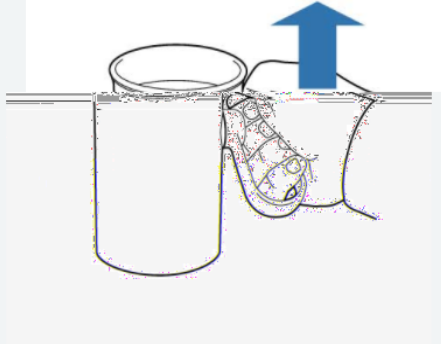


Hold the handle at any angle and the cup stays upright

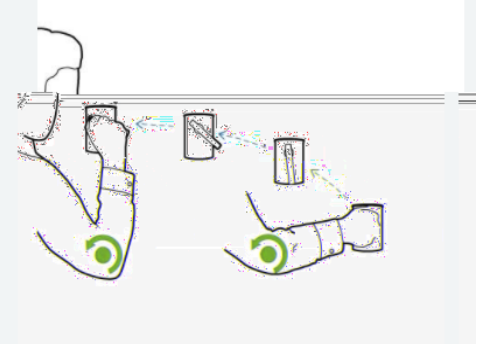


Bring to your mouth without needing to bend your wrist

Step 2. Tilts Easily


You can easily tilt handSteady® cups to drink with more comfort without needing to bend your wrist, raise your elbow or lift your head back as far.

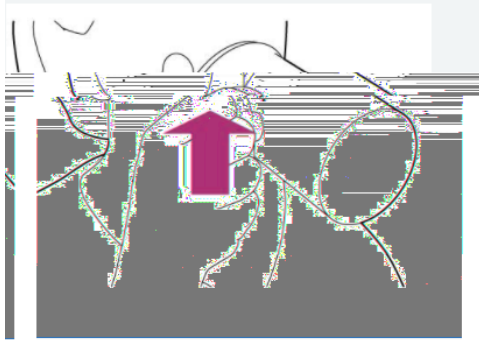

Use your mouth to pivot the cup to tilt it

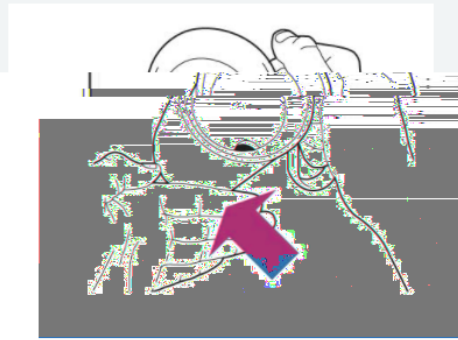


Use your other hand to pivot the cup to tilt it

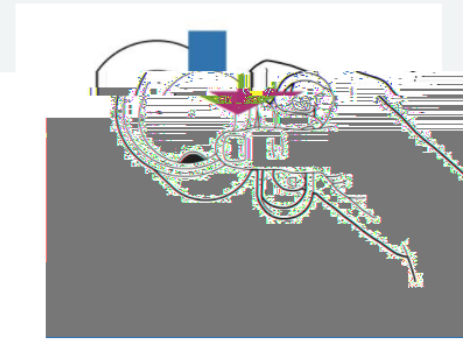


Use your thumb to pivot the cup to tilt it

Step 3. Keeps Steady


handSteady® cups are designed to be more steady in a several ways if you have tremor or reduced coordination. This reduces spills, saving time and money, reducing anxiety and increasing control, well-being, confidence, independence, inner peace and dignity.

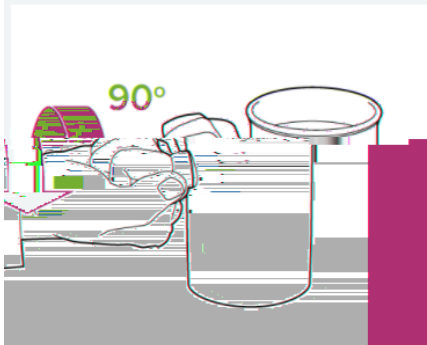

Hold the handle in stable positions where tremor is less

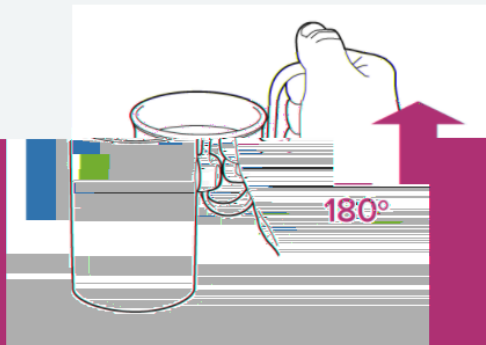

Hold the handle at 180° for increased stability

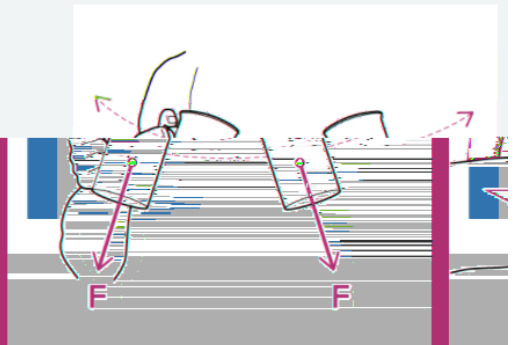

Let the cup swing freely and centrifugal force stops spills