

### The NEW Matrx® E2™ Back Series has taken back support to a new level!

- Hardware Updates
- Ripple Foam Technology
- Standard 3" Contour
- New Accessories
- Enhanced Immersion
- Deep 6" Contour



## What's New



### RIPPLE FOAM

A leap forward in temperature and moisture control, immersion and comfort

#### **BACK SHELL**

More cut-outs for breathability and the ability to add accessories

#### **MODELS**

Standard 3"contour depth, and deep contour 6" contour depth

#### **ACCESSORIES**

New hardware updates and new innovative accessories such as BackBone PSIS support, PostureFlex abdominal support, Matrx Protect Slipcovers for advanced infection control, and many more.

# Ripple Foam


Temperature and moisture control, immersion and comfort.

- Airflow channels for temperature and humidity control
- Rippled top surface for immersion and comfort, solid HR foam backing for support and protection
- Extra immersion and shear reduction in spinal relief area
- Allow perfect fit of foam to back contour







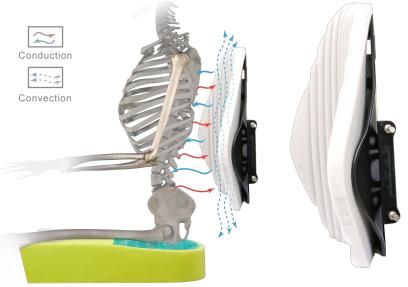

## **Back Shell**

- Available in two contour depths for midline support 3" and 6"
- Weight Capacity: 133kg.
- Three holes for attachment of PostureFlex Abdominal Support at desired position
- Notched cut out for sacral area
- Holes provided for attachment of Headrest and Chest Harness Interface on all backs 14" and higher

## **E2 BACK**

## **E2 DEEP BACK**







			WIDTH		
		14"	16"	18"	20"
неіднт	Fits W/C	13"-15"	15"-17"	17"-19"	19"-21"
	10"	*	*	*	
	12"		*	*	*
	16"	*	*	*	*
	18"	#	#	#	
	20"		#	#	#
*					









## **Accessories**

#### **EASYFIX LATERALS**

- Fixed laterals can be installed inside or outside of the back shell
- Height adjustable in vertical slot
- Compatible with E2 standard contour backs 10" and higher







#### **SWING AWAY LATERALS**

- Standard or offset hardware available
- Compatible with E2 standard contour backs;
   Standard 10" and higher; Offset 12" and higher
- Available in 2 pad sizes:

**S**: 5.25" L x 3.5" H **L**: 6.5" L x 4.25" H

Standard

Offset



#### MATRX PRIVACY FLAPS

 Attach to loop strip on E2/E2D backs

Available in 4 sizes:

XS: 14" W

**S**: 15" &16" W **M**: 17" & 18" W

L: 19" & 20" W



#### **HEADREST INTERFACE**

- Moulded lightweight, low profile mounting plate for headrest attachment
- Included with all backs 14" and higher
- Easy to install without removing back cover

**E2HDI** Heavy Duty option with extra strong steel headrest mount for extreme tone or heavy use

#### POSTUREFLEX ABDOMINAL SUPPORT



- 3 mounting heights available in pre-drilled holes on back shell
- Width and tension adjustable hook and loop closure
- Easy keyhole attachment
- Designed to provide comfortable upright postural support



Available in 2 sizes: **W**: 20 - 32" L x 7" H

M: 16 - 23" L x 6.5" H

### QUAD RELEASE TAB FOR POSTUREFLEX

Quad release tab for easy independent application and removal of PostureFlex



#### **BACKBONE**

- Contoured anatomical PSIS/ Sacrolumbar Support
- Can be trimmed to desired size
- Loop velcro molded into foam for ease of adjustment



### COVER

- Moisture resistant and breathable Meshtex<sup>™</sup> outer cover
- Foam and cover overlap the shell for protection and comfort



# **EasySet**

## **MiniSet**

Adjustments: – Angle (40°), depth (2" in front or behind back canes), can be offset for rotation. Fits: – 1" back canes, inserts for 3/4" & 7/8 " back canes include



- Standard on all E2/E2D backs 18" and 20" high
- Quick release



- Mini version of the EasySet
- Standard on all E2/E2D backs up to 16" high
- Quick release

Optional

#### Optional -









Fixed EasySet Recommended for tilt/recline applications Requires tools for removal

Heavy-Duty EasySet Recommended for users with high tone or heavy intended use Quick release

Split

Allows clearance of frame obstacles, quick release Compatible with EasySet Pin Interface only





Fixed MiniSet
Low profile design
compatible with rigid
chairs, recommended for
tilt/recline applications
Requires tools for removal

Heavy-Duty MiniSet
Recommended for
high tone or heavy use
applications
Quick release



#### **MOUNTING PINS**

- Height and width adjustment with user seated in the back
- Hardware can be positioned to avoid frame obstacles like arm supports and rigidiser bars
- Fits chair width and 1" wider or narrower

### EasySet Pin Interface Standard on 18" & 20" back heights

MiniSet Pin Interface Standard on back heights up to 16"



#### **CHEST HARNESS INTERFACE**

Allows precise placement of shoulder strap guides for most effective positioning



**E2CHI-S:** 14" to 15"W backs **E2CHI-M:** 16" to 20"W backs Enquire for availability

### **MATRX PROTECT SLIPCOVER (optional)**

- Moisture resistant Startex™
   Slipcover compatible with disinfecting wipes
- Welded seams for advanced infection control
- Easy to apply and remove for laundering
- Enquire for availability



SOUTH

Ph: 0800 212 333 E: info@c1south.co.nz www.c1south.co.nz